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We would like to present the ideal complement for any aspiration unit: The MESTRA® "Ciclón" Dust Decanter. This machine will allow you:

- To pre-filter the dust and dirt aspirated with a high efficiency (99%) and save a considerable amount of filters for your Aspiration Unit.
- To reduce the loss of charge caused by the clogging of the Aspiration Unit filter. With the "Ciclón" Dust Decanter dirt gathers at the bottom of the machine, so allowing a constant airflow without obstructions.
- To reduce the fatigue of your aspiration unit. The airflow does not decrease as the filter fills up, and therefore the stress of the motor decreases, and its cooling and working life improve.
- To prevent the clogging of the filter caused by humidity when working with wet materials (plasters, etc).
- To recover the dust aspirated in a clean and simple way. This is especially interesting when working with precious metals.

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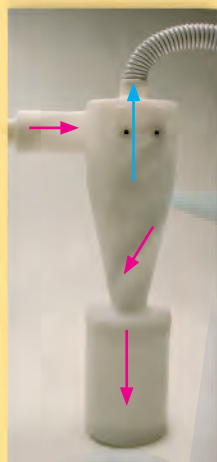
The installation of the unit is very simple. It is simply fixed to the wall or a holder with three bolts. The intake pipe can be positioned to either side, left or right. The unit is factory supplied with all the accessories required for its complete installation: connecting hoses, intake adaptors, holding plate, fixing bolts.

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The operating principle of the "Ciclón" Dust Decanter is a very ingenious one: the aspirated airflow circulates within the machine's body along a helicoid. The own weight of the dust prevents it from moving up toward the outlet pipe and makes it fall to the tank.

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The "Ciclón" Dust Decanter features a high capacity tank (3.5 L). Emptying and cleaning the unit only takes a few seconds, due to its excellent accessibility and simple threaded lid.

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The "Ciclón" Dust Decanter works perfectly together with different machines and in different situations, from a simple workbench to a sandblaster. Its small size and compact design allow it.

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