MESTRA®

**MESTRA®**

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Talleres Mestraitua, S.L. **MESTRA®** is a Spanish company specializing in manufacturing dental machinery and tools. From our foundation in 1945 we have kept a steady expansion line. There are many factors contributing to this dynamic, but we should mention three of them as the most important ones: a strong and professional relationship with our clients, the high quality of our products, and the constant innovation in our designs.

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**The design department is considered by the company management as a key element to develop its strategy. Besides, a team of experienced and highly skilled engineers is in charge of developing the new projects.**

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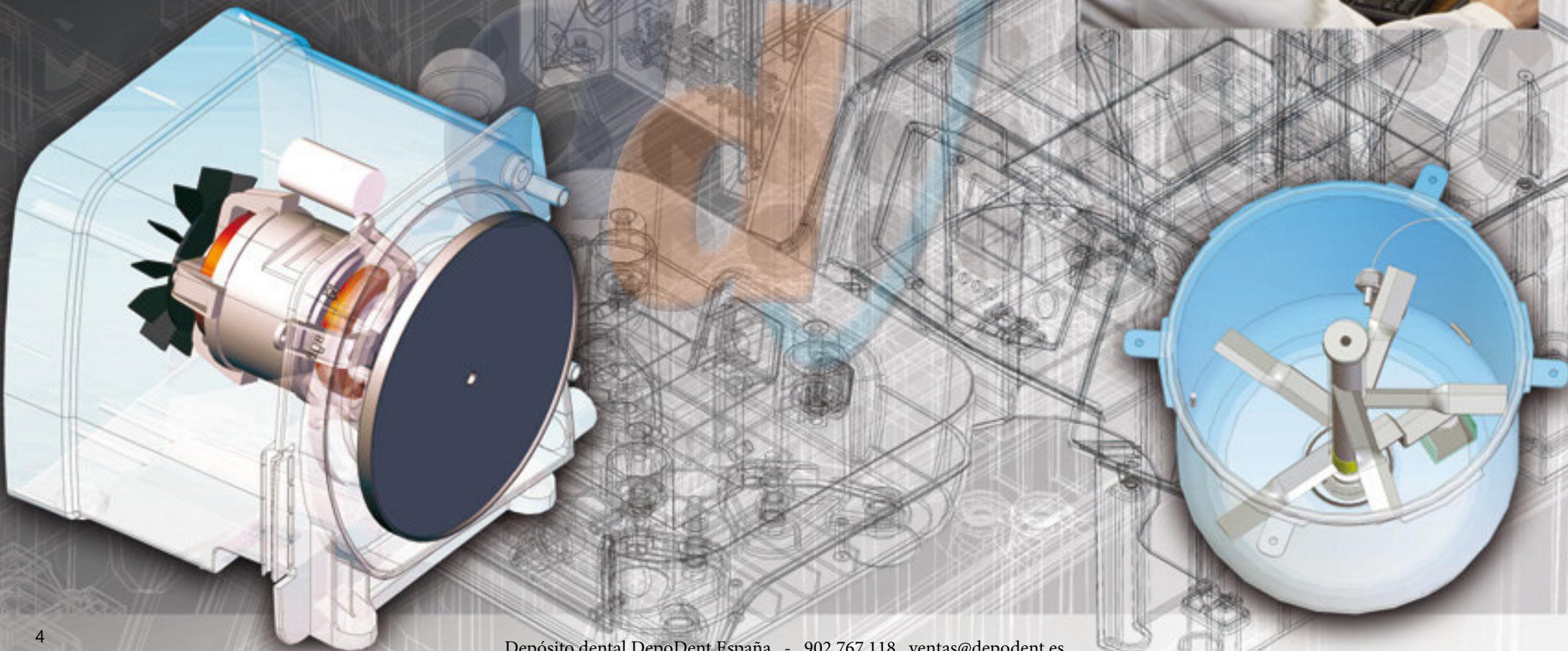
**MESTRA®** invests a large proportion of its resources in research. Product innovation and providing smart technical solutions to the daily problems of the dental technician are our main challenges.

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The **MESTRA®** products have a reputation for being robust and reliable, and for having the most important worldwide certifications. In regard to value for money, we are one of the best options in the market.

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The Product has to be very reliable and durable. Besides, every machine should be repairable (the "if it fails, throw it away" concept is not accepted) A quality product has not to be necessarily expensive. Its price should be reasonable, competitive and according to its performance.

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Nowadays **MESTRA®** exports to more than 50 countries in five continents, and offers a wide catalogue with more than 1000 different products. But its most valuable resource is its human team, composed of highly skilled professionals.

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In **MESTRA®** we are fully aware that many of our clients are far away from our company. The remote service is our strong point. Our technical support is one of our most satisfying sections, the service we are most proud of.

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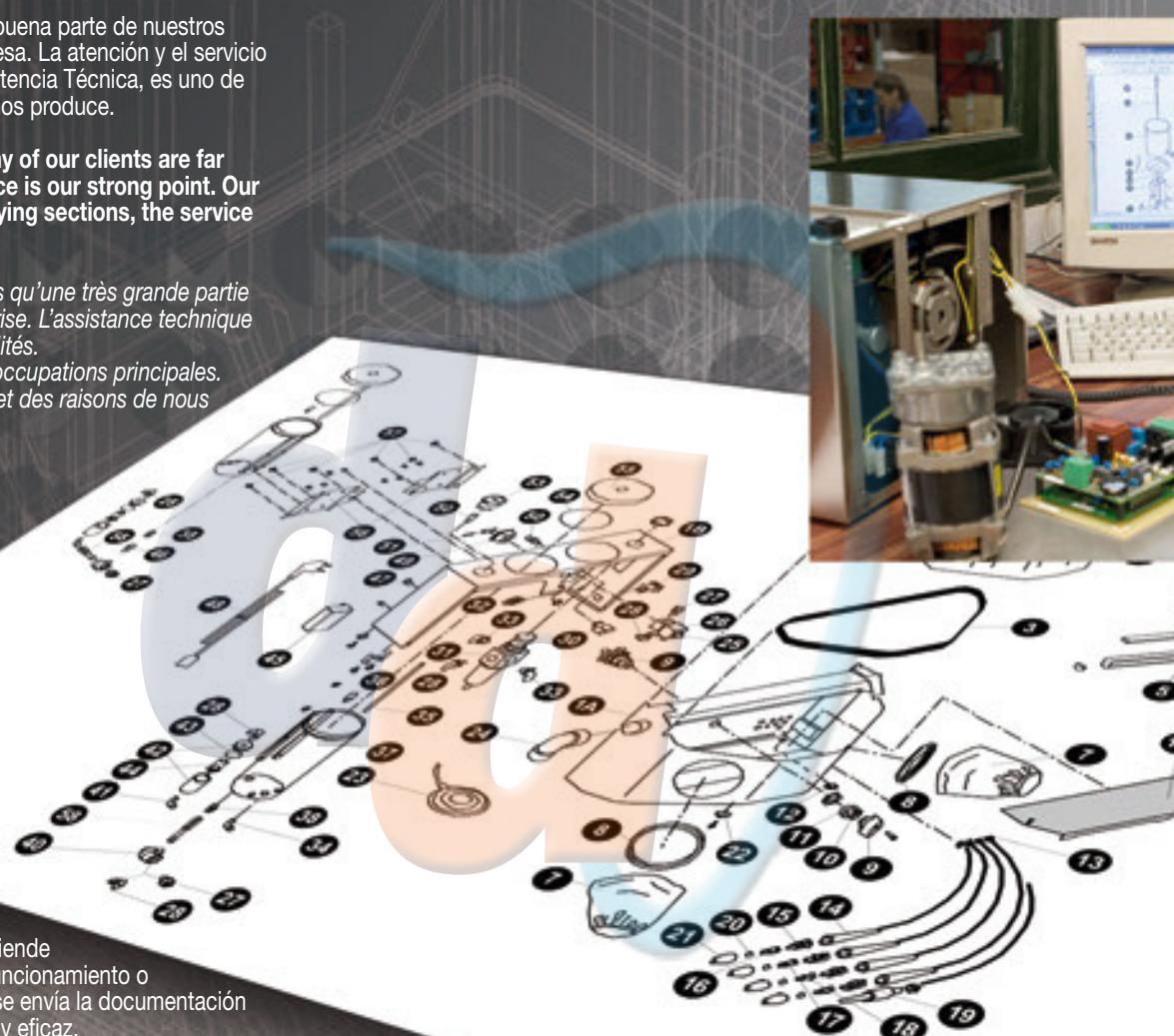
Our technicians in the Technical Department will help you personally clear up any doubt you may have about the product's operation or repair. When needed, we will have the required documentation sent to you. Our answer is quick and efficient.

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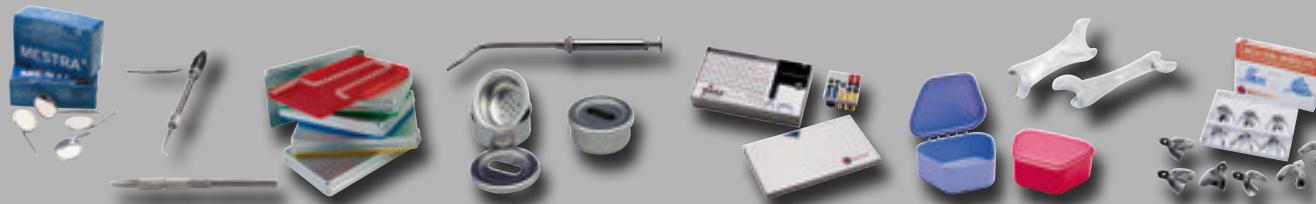
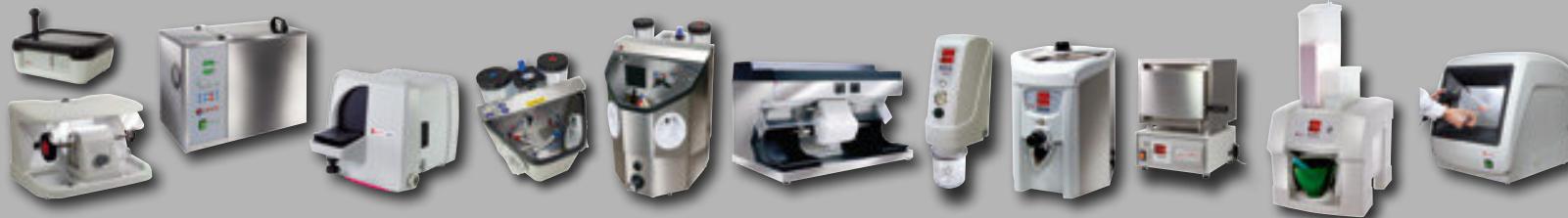
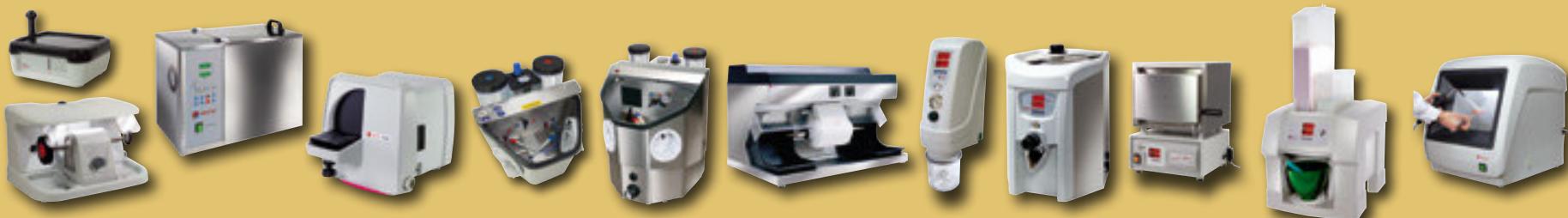
In MESTRA we opt for designs with their own personality, which makes them distinguishable from our competition's products.

Taking into account the final output, these three aims have been completely fulfilled in the three interesting products we have marketed:

1. The Teide 1 vibrator, it has a simple vibration frequency with adjustable power. Simple but efficient.
2. The Teide vibrator, featuring an electronic card that allows adjusting the power and two types of vibration.
3. The Teide Cima vibrator, similar to the previous models, but its electronic card allows –apart from adjusting the power– selecting five different types of vibration and adjusting the vibration time.

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Details are without a doubt the most clear distinction between high quality products and average quality ones. This is why in MESTRA we pay them a special attention. The basic shapes have been designed to prevent plaster debris getting inside the device, since the working environment of the vibrator tends to be specially dirty. The table rubber edges (1), the embedded controls (2), or the rounded corners (3) are details that prevent dirt getting inside the device.

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The table's dimensions have been thoroughly calculated, trying to fit as many cylinders of different size as possible on the smallest possible surface. After a complete combinative work, our engineers concluded that the best combination was a 220x180 mm table.

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The legs are an important element to reduce vibrations. Traditionally the vibrator legs were equipped with suction pads to fix the legs to the table, or designed as highly flexible hollow pivots. Our engineers in MESTRA succeeded in joining both approaches of hollow legs with a suction pad at their end. The result of this ingenious solution is astonishing: the vibration and noise transmitted to the table are drastically reduced.

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The multiple ribs and supports give the frame an exceptional stiffness, similar to the stiffness of those made of steel plate. In addition, it is built in one single piece, which makes it even stiffer. The result is an extremely smooth operation and a very low transmission of vibration and noise.

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The pouring stand is based in a completely innovative concept. Traditionally most manufacturers have designed this element based in the same basic concept: a rubber ball attached to the vibration table by a retractable rotating arm.

Nevertheless, our engineers in MESTRA opted for a completely innovative approach: a spike-shaped stand screwed to the table.

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# Máquinas Machines Machines

**ELECTRO VIBRADOR GRANDE "CLASSIC"**  
**(CLASSIC) LARGE ELECTROVIBRATOR**  
**VIBRATEUR ELECTRIQUE GRAND (CLASSIC)**

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**Made of steel plate. Epoxy paint. Equipped with illuminated start and stop switch. Potentiometer for progressive adjustment of vibrations.**

**Resting on suction pads. Working plate made of rubber on a metal body.**

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**Its characteristics are similar to those of the Square Electro Vibrator, except for its dimensions: 220 x 160 mm.**

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Its body is made of white colour anodized aluminium, and rests on suction pads. Working surface is made of rubber on a metal body and has a recess of 30 mm for inserting the tray. It is equipped with a potentiometer, selector switch for stepped adjustment of vibrations and "ON" light.

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# Máquinas Machines Machines

LAVA-POLIMERIZADORA E2 A. INOX.

E2 WASHING AND POLYMERIZING MACHINE INOX.

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The E2 washing and polymerizing machine is equipped with a processor to control the temperature according to two set programmes.

1.º Programme 1 (shower programme)

- a) The operator or user can set a temperature (e.g. 80 °C) so that the E2 washer/polymerizer is ready to shower at all times.
- b) Shower time required.

2.º Programme 2 (polymerization programme)

- a) Delayed start time in hours and minutes from 0.00 to 99.59. This means the programme can be set to run at a particular time. This delay cuts in each time the programme is started.
- b) Duration of each controlled temperature increase gradient in hours and minutes from 0.00 to 99.59.
- c) Required holding time at each temperature plateau in hours and minutes from 0.00 to 99.59.
- d) Shower time required.

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**The washing system with 6 high-pressure water injectors ensures proper scalding of flasks and perfect wax removal.**

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**Large compartment with room for 6 clamps with 2 flasks each.**

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**Fast spray tree assembly & disassembly system.**

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Completely made of stainless steel. Together with the Washing and Polymerising Machine R-080060, they make a practical, aesthetic and unchanging combination in dental labs. It has three inner shelves in order to keep adequately flasks, clamps and any other material used in polymerising processes.

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#### M-9 POLYMERIZER FOR FLASKS

Made completely of stainless steel.

Polymerizes 9 flasks with clamp 030390

The M-9 polymerizer is equipped with a processor to change temperature according to a pre-set programme.

On the M-9 the operator (user) can select the following:

- a) A delayed start time set in hours and minutes from 0.00 to 99 hours 59 minutes. This means the programme can be pre-set to start at a specific time. The delay works every time the programme is initiated.
- b) The duration of each temperature increase gradient in hours and minutes from 0.00 to 99 hours 59 minutes.
- c) The desired holding time in hours and minutes from 0.00 to 99 hours 59 minutes at each plateau.

#### END OF TREATMENT WARNING BUZZER

- When the programme completes treatment or A4 a modulated warning buzzer is set off for one minute. When the alarm buzzer sounds, press any key to acknowledge the alarm and continue the programme.
- If the power fails the regulator stores all its data, and when power returns it continues from the same point in the programme.
- Programmes and parameters are stored indefinitely when power to the regulator is cut off.

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Room for 3 clamps with three flasks each.

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It is exactly the same as the R-080400 except for its size that can hold up to 18 flasks.

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### MICROPROCESSOR

- Time & temperature control.
- Buzzer to indicate end of program.
- Last program memory.
- Automatic disconnection.
- Triac for heating control
- Thermocouple breakage indicator.
- Protection against voltage variations
- Protection against electrical power cut.

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R - 080406

LAVA-POLIMERIZADORA E2 *Concept*

Concept E2 WASHING AND POLYMERIZING MACHINE

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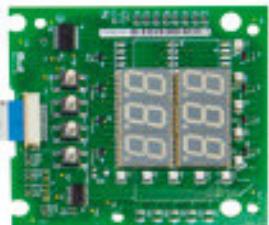
The "Concept" controls family has been developed as an economical alternative to the MESTRA's standard electronic controls. Their performance and features meet perfectly the needs and expectations of most of the users, but at a very appealing price. And this is why they are particularly recommended for clients who have not very special operation requirements. Apart from the price, another advantage is their easy operation and programming). The "concept" line is available for the M9 and M18 Polymerizers and for the E2 washer-polymerizer.

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## MICROPROCESSOR

- Time & temperature control.
- Buzzer to indicate end of program.
- Program 4.
- Automatic disconnection.
- Triac for heating control
- Thermocouple breakage indicator.
- Protection against voltage variations
- Protection against electrical power cut.

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## FEATURES:

- a) Delayed start time from 99h. 59 min.
- b) Time taken to reach temperature or holding time from 00.00 to 99 h. 59 m.
- c) Water temperature (max: boiling).
- d) Buzzer and automatic disconnection.

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## Room for 3 clamps with three flasks each.

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Modern and versatile model trimmer. It features a powerful 1600 W engine with integrated brake, opening lid for easy access to the disc, operation warning light, discs with hook-and-loop type (Velcro®) fastener, sophisticated cleaning system, adjustable worktable... and an endless list of elements which make it –probably– the most complete model trimmer in the market. Unbeatable quality/price ratio.

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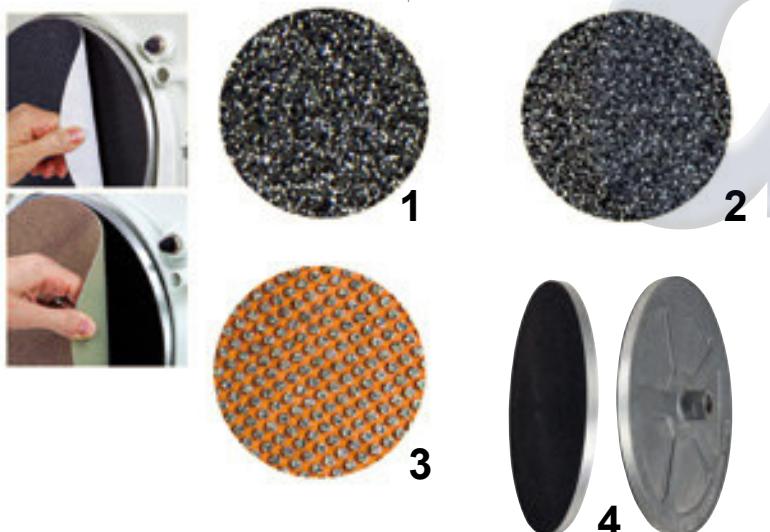
## CAUTION!

Diamond discs are very heat-sensitive: do not trim or  
rectify metal pieces or coatings.

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Easily adaptable to the R-080085 and  
R-080087 decanters.

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The MESTRA model trimmer is a strong and robust machine, featuring the latest improvements and innovations required by a machine of these characteristics. Its robustness can be seen at a glance and its weight of more than 21 Kg (maximum on machines of this type) added to the use of non slip rubber feet assures perfect stability during facing operations. It runs noiselessly as the special resin wheel is carefully balanced.

The wheel is mounted on an aluminium plate, thus achieving a larger working area as it has no shaft or external nut hinder grinding operation in this area. The motor is provided with guards, giving a more compact and harmonious appearance, while ensuring safe operation, preventing any possible accident. The MESTRA trimmer is equipped with an electrovalve for electronic feed of water by pressing a switch. This switch is protected with a PVC insulating cover to prevent any electric shock due to humidity. It is also provided with an earth cable. Water flow is directly adjusted from the machine through an adjustment valve. The drainage system is of lateral type, designed to prevent clogging as drain hole is oversized 30 mm and the water carrying plaster particles is driven outward by air turbulence generated by the wheel.

The component supporting the patterns is of tilting type with three positions. The motor is monophasic, and is insulated from the wet area of the machine by means of a retainer with a stainless steel spring for a perfect fit on motor shaft end. The motor shaft is made of hardened and ground stainless steel. The MESTRA model trimmer is entirely made of aluminium excepting a few components which are made of stainless steel. The machine is painted with epoxy paint, and always looks as good as new.

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**Large, 120 mm working window to facilitate the cutting of large models, enabling better use of the disc.**

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**Easy-to-extract, 3-position, fold-down table (80, 90 & 100°). Marked for referencing trimming on precision models.**

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**Front-mounted water flow adjustment tap.**

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**Large, side-mounted drainage system. Water with plaster is expelled by the turbulence caused by the grinding wheel.**

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**Electrovalve for automatic water inflow when the main switch is engaged.**

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**Safety switch: if electricity fails the RH3 is disconnected and must be manually reset (thus preventing accidental start-up).**

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### Classic series model trimmers

**Family of high performance model trimmers, designed for intensive use in the dental lab. Reliability, robustness and durability are their best presentation. These trimmers can stand the most demanding and hardest working conditions. A strong aluminium chassis guarantees their robustness, easily noticeable at first sight. The total weight of the machine is 21 kg, which guarantees its stability during the trimming. Their operation is smooth and very silent, due to the thorough balancing of the disc. They are precision machines with which you can carry out trimming works quickly and easily. The trimmers of the Classic family are an excellent option for those professionals looking for a long lasting machine, of low maintenance and bomb-proof reliability. In short, a family able to meet the expectations of the most demanding professionals for many, many years.**

### Features of the family:

The disc is well balanced, which guarantees a completely smooth operation. Vibrations are unnoticeable. The motor is encapsulated, which prevents dust and dirt getting in and extends its working life. It also prevents water getting in in case of water leaks. A high resistance aluminium chassis gives the machine weight and a better stability while trimming. The machine is factory supplied with its accessories: a rubber pipe, a spanner to release the disc

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### 1500 rpm silicon carbide disc trimmer

**Due to its low speed, the main advantage of the trimmer is its easy handling, its silent operation and the scarce splashing it produces. Thus, it is recommended for professionals not very familiarized with trimming who work in places that have to remain clean or in places where they cannot make too much noise.**

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### 3000 rpm silicon carbide disc trimmer

**It is the model that best meets the requirements of the dental lab. Its trimming power is very high, and it can cope with a heavy workload in a very short time. Recommended for experienced professionals working in places where a little more noise or splashing is not a problem.**

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### 1500 rpm diamond disc trimmer

**Similar features of the previous trimmer, but with a diamond disc. The diamond disc, if used properly, has a much longer working life. Besides, the trimming surface is maintained completely flat, with no wear-scratches. Its price, however, is higher.**

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### 3000 rpm diamond disc trimmer

**Identical features of the previous model, but with a diamond disc.**

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## CAUTION!

Diamond discs are very health-sensitive:  
do not trim or rectify metal pieces or coatings.

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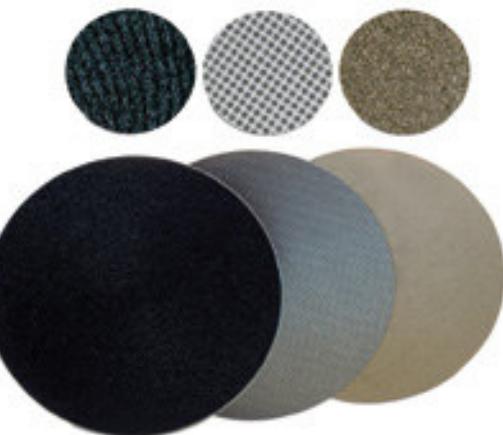
The main characteristic of the RDS-1 trimmer is that the surfacing operation is performed dry, with a diamond disc at 1.500 r.p.m.  
-Stability of the material to be trimmed, as it is not wet.  
-Higher performance and longer life, minimal effort.  
OPTIONAL EXHAUSTER (ESSENTIAL).

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## CAUTION!

Diamond discs are very health-sensitive: do not trim or rectify metal pieces or coatings.

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Made of resistant polypropylene. It has been specifically designed for a quick and easy cleaning, since dirt is collected in a plastic bag. The lid closes hermetically and prevents the spread of bad smells. Its size and shape have been designed for the unit to be easily installed under the sink.

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**It is made in highly resistant fibreglass with a total capacity of 70 l. and a design that allows an easy installation under sink. It also has wheels which make its movement easier. Cleaning and emptying operations are easy because of the optimum access to the cube. This machine is supplied with all couple nuts and accessories for its installation.**

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# Máquinas Machines Machines

SANDBLASTER PLANETARIUM PLUS IV (CON CALEFACTOR) R - 080246

PLUS IV PLANETARIUM SANDBLASTER (WITH HEATER)

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High-range sandblasting machine. It has four pots, each one with an independent pressure gauge and pressure control system, which provides great accuracy in fine blasting. A heating system located in each of the pots keeps the sand free of humidity, thus avoiding the appearance of lumps in the nozzles.

The working cabin has been carefully designed to achieve a comfortable and ergonomic working position. The interior is lit with a fluorescent lamp and does not have shadow zones. The machine is supplied with tungsten carbide nozzles of 0.75mm.1 mm. and 1.25 mm.; a control pedal; and in an interior air jet, which is very useful for cleaning the inside of the cabin. Plastic protectors for the glass are available (R- 080242-50).

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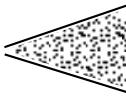
**Working chamber is large, ergonomic and very well illuminated. Tungsten nozzles of 0,75, 1 and 1,25 can be fitted into the guns. An air nozzle, highly practical when cleaning, is also available. The chamber valve is inside the chamber. The colour criterion allows the right chamber to be identified and selected. A rubber attachment inside the chamber may be used for initial adjustments of the sand jet.**

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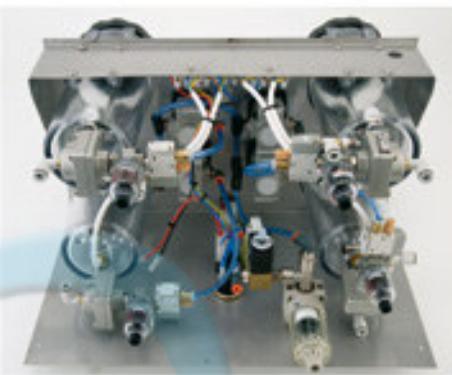
**Each chamber has its own mano regulator and manometer. At the central part of the machine there are the electronic control: a switch for illumination of interior and a switch of the sand heating resistors.**

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**Electrical and pneumatic circuits are made of high quality materials: electrovalve with mobile nucleus and valves of quick discharge that immediately cut the flow of air saving sand. Observe the precision and smooth action of the 5-way distribution valve.**

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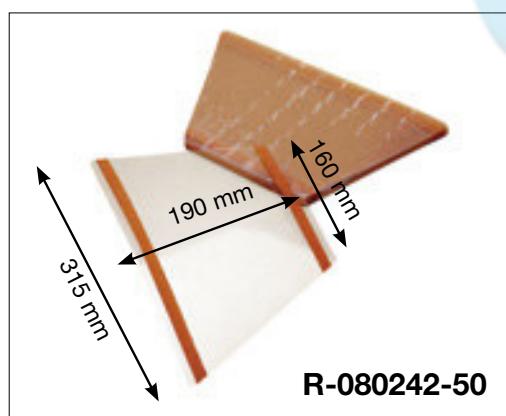
**Air is filtered before entering the pneumatic circuit. Filter drip trap has an easy access and it can be automatically operated.**

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**Easy to place self-adhesive Protective Covering. It prevents glass from wearing out due to sand abrasion. Packs of 10 units.**

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The Constellation family comprises a range of modular sandblasters that are able to adapt to the needs of most professionals. Their models have 2, 3 and 4 sand chambers so this machine is a very versatile tool in Dental Lab.

The main characteristics of the "Constellation" Sandblasters are the following:

- Stainless Steel Chassis AISI 304 with a broad glass window.
- Blowing gun inside the chamber, very useful when cleaning.
- Lighting system with LED
- Modular system, so it can be extended in the future
- Chamber changing valve inside the chamber
- Relief valve that stops air flow when the pedal is released, so sand is saved
- Ergonomic pedal connected to the machine by flexible tube

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**All Constellation sandblasters are built completely in stainless steel, a noble material which is easy to clean, and will not rust.**

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The "PAINTED" model differ from the standard one in its chassis, made in this case of a painted steel-sheet, an interesting option for those users who look for a high performance sandblaster at a reduced price.

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**The blasting chamber shows operations clearly through the glass. Both the user's hands and back are in a comfortable position, thus reducing fatigue.**

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**The Constellation family has a modular design: the basic two-chamber model can be extended for 3 and 4 chambers. Since the system is so simple to use, the user may extend the models with no assistance.**

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**Top-of-the-range components have been used in the pneumatic circuit. Observe the precision and smooth action of the 5-way distribution valve.**

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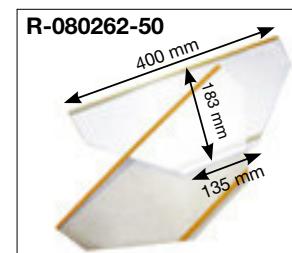
**To prevent contamination within the pneumatic system, the Constellation range has an ingenious filter-discharge-non-return system for safe use of sand up to 25 microns.**

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**Self-adhesive plastic protection. It is easy to place and avoids glass wearing due to sand abrasion. Supplied in packs of 10 units.**

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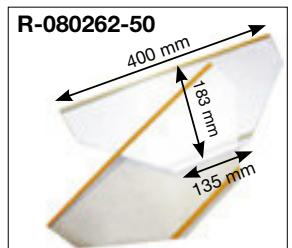
# Máquinas Machines Machines

CHORRO DE ARENA "CONSTELLATION" PRACTIC (pintura epoxi) R - 080259

PRACTIC "CONSTELLATION" SANDBLASTER (epoxy paint)

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**NEW**

PLASTICO PROTECTOR PARA CHORRO DE ARENA "CONSTELLATION" R-080262-50

PLASTIC PROTECTION FOR CONSTELLATION SANDBLASTER

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Excellent option for those looking for the performance and reliability of the Constellation sandblaster family at an unbeatable price. Its design has been optimized to offer a product of an optimum value for money. It has two sand chambers, LED lighting, ergonomic vase changing system, wide interior chamber, excellent sight through a broad glass window, top quality nozzles, etc.

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# Máquinas Machines Machines

CHORRO DE ARENA "PLUTON" ACERO INOX

STAINLESS STEEL PRACTIC "PLUTON" SANDBLASTER

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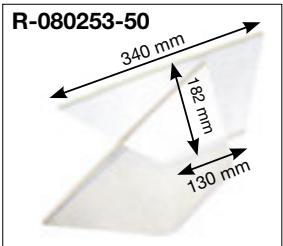
Plutón: mini size; gigantic performance.  
Small sized, but with similar features to those of the bigger models.

- Two high capacity tanks
- Blasting pressure up to 6 bars
- Stainless steel body
- High power LED light
- Wide window

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## Size does matter...

The arrival of new technologies and the increasing sophistication of the techniques have finally made space one of the most valuable resources among professionals in the Dental Sector. The only way to arrange so many different machines and devices is to reduce their size. Nevertheless, reducing the size does not mean simplifying: the aim is to achieve small machines, but with the same or even better performance than their elder sisters.

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# Máquinas Machines Machines

CHORRO DE ARENA "GALAXY"

SAND BLASTER WITH STATIC GUN AND BASKET

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All Galaxy sandblasters are built completely in stainless steel, a noble material which is easy to clean, and will not rust.

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Sandblaster totally made in Stainless steel. It has a removing spinning basket for the automatic skeletons blasting, and a 3 mm. diameter static nozzle. In the automatic performance mode, the basket spins and the nozzle blows for a time —set by the user— between one up to 30 minutes. Also it is possible to blast in a manual mode just pushing a button. The sand tank is over-dimensioned to load a big quantity of sand, enlarging this way the automatic cycle time. The wide window allows a great vision of the chamber inside. The glass is protected against abrasion by a plastic coat. The machine is supplied with a Boron Carbide nozzle (C4B) a material with more much abrasion resistance than the traditional tungsten; and with two plastic glass protective foils.

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**The pieces are placed in a motor-driven spinning basket.**

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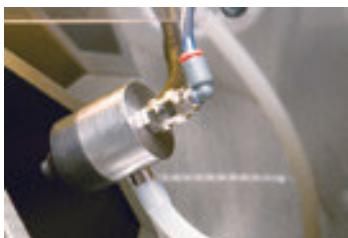
**The blasting nozzle can be positioned to face the three space dimensions, so an optimum blasting angle can be always achieved.**

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sistema neumático de tipo Venturi.

**The sand flow moves along a closed circuit pushed by a Venturi-type pneumatic system.**

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**The Galaxy Sandblaster with Static Gun features a button for manual operation. This way, after the automatic sandblasting, the last finishing touch can be easily given by properly positioning the nozzle.**

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**The basket can be easily disengaged from the motor shaft, which makes preparing the works easier.**

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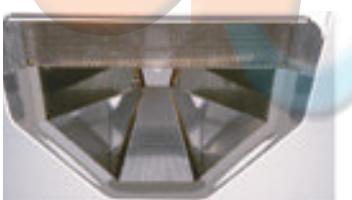
**The sand in the hopper deposit is absorbed by the Venturi aspiration and then it is pressure-injected by the nozzle. Then it falls again into the hopper deposit.**

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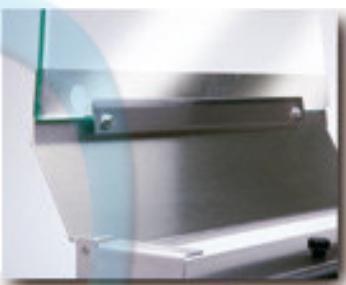
**The sand hopper deposit has a capacity of 12 Kg. Its volume, along**

**with its great depth and its steep walls, allows to operate the machine in full automatic mode for more than 30 minutes.**

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**The glass of the upper window is doubly protected: it has a protection coat of synthetic resin, and a disposable plastic protection.**

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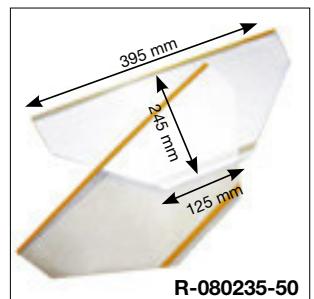
**The Eolo Plus Aspiration Unit, the Cyclone Dust Decanter, and the Galaxy Sandblaster with Static Gun have been specially designed for an optimal performance working together, creating a blasting-filtering-aspiration set extraordinarily efficient in professional environments.**

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**Self-adhesive protective covering.  
Packs of 10 units.**

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Based on the popular MESTRA's Galaxy model, the COSMOS sandblaster includes two micro-blasting nozzles similar to those used in the Constellation model, becoming a unique and powerful device with a Ø 3 mm fixed nozzle and micro-blasting tools. You save the space and money required to buy two different devices separately. It is completely made of stainless steel. Factory supplied with a Ø 3 mm boron carbide fixed nozzle, two tank/nozzle sets and a set of glass-protection covers. Ergonomic pedal and blowing nozzle.

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The cabinet has to be of the right size. If it is too big, it takes too much space. If it is too small, it will be difficult to place the pieces and work comfortably. The size of the Cosmos has been chosen to achieve a perfect balance between space and ergonomics. Its width, the working height, the depth of the chamber... have been thoroughly analysed.

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**The sand recirculation system also has its own peculiarities.**

The hopper has to be narrow and deep if we want to ensure the continuous operation of the machine with reasonable amounts of sand, and always maintaining the aspiration hose covered. This has probably been the feature most deeply analysed by the **MESTRA** design team while designing the static sandblasters. The result is obvious: the Cosmos sandblaster can operate continuously and automatically for several hours with only 4 kg of sand.

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The Cosmos Sandblaster with Static Gun is emptied by simply removing a drain plug placed at the bottom of the machine.

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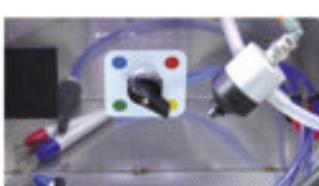
The lighting of the chamber is made by energy saving LED lamps with a colour temperature of 5600-6000 K, the most similar to sunlight that can be achieved with artificial lighting.

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**The chamber-change valve is inside the chamber. The colour criterion allows the right chamber to be identified and selected. A rubber attachment inside the chamber may be used for initial adjustments of the sand jet.**

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**The controls have an ergonomic arrangement. Operations such as filling the chambers with sand or adjustment of pressure levels are easy to carry out.**

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**The Eolo Plus Aspiration Unit, the Cyclone Dust Decanter, and the Cosmos Sandblaster with Static Gun have been specially designed for an optimal performance working together, creating a blasting-filtering-aspirating set extraordinarily efficient in professional environments.**

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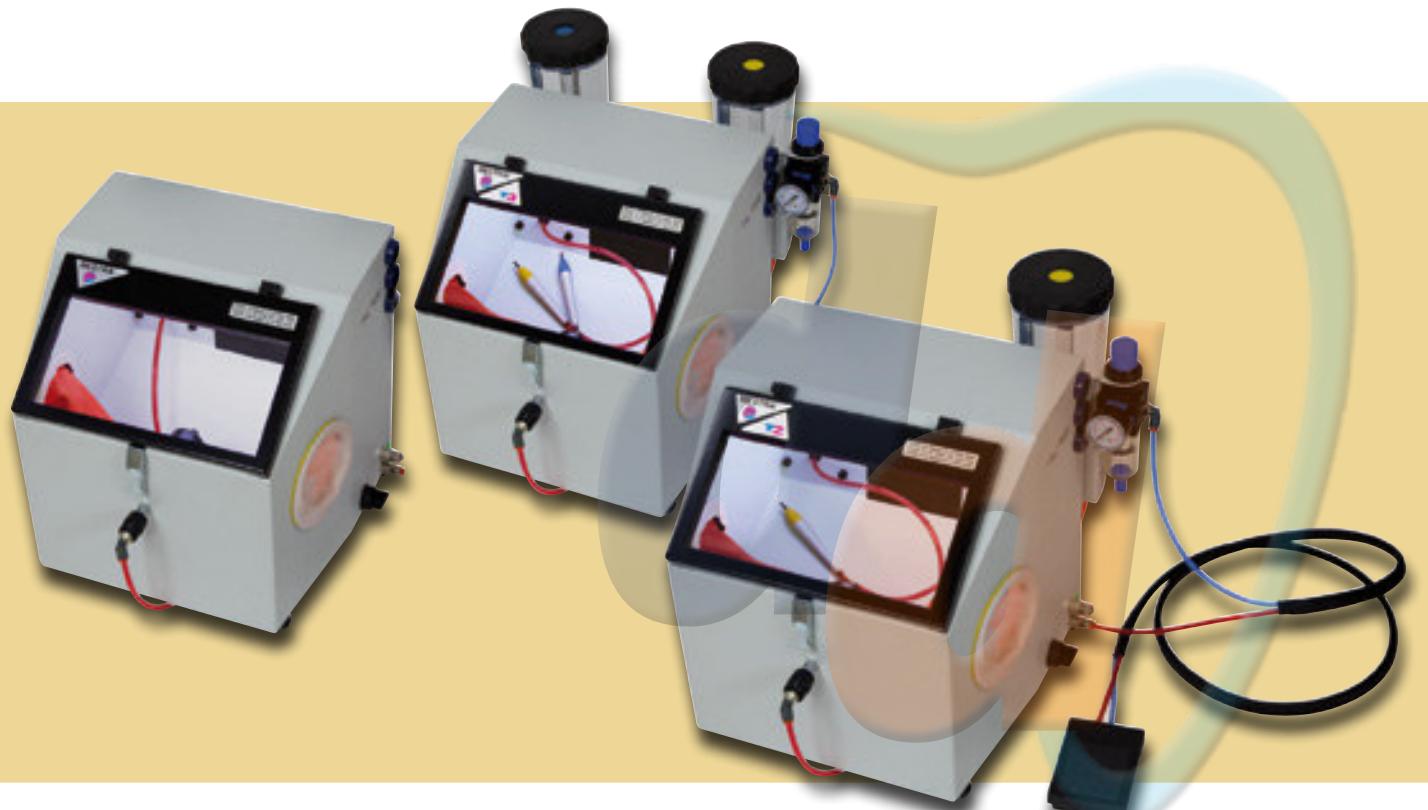
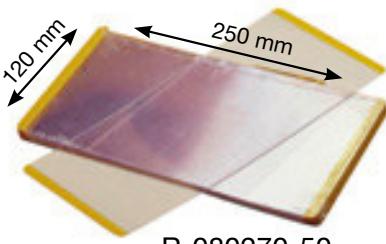
**Self-adhesive protective covering.**  
**Packs of 10 units.**

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Similar features to those of the T1-T2-T3 well known models, but with a major difference: TERRA sandblasters have a bigger chamber and a more powerful LED lighting. Besides, the position of the holes to insert the hands has been modified to allow a more ergonomic operation.

The name of the three models is still similar:

**TERRA I** It features just a fixed nozzle

**TERRA II** It features a fixed nozzle and a cup for micro-blasting

**TERRA III** It features a fixed nozzle and two cups for micro-blasting.

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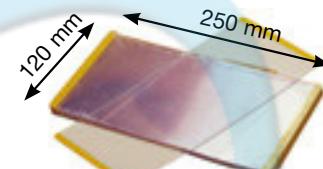
**This is a compact and economic family of Sandblasters. All of them are equipped with a fixed nozzle of 3 mm diameter with sand re-flow system. The T2 model also has a 1,25 mm mobile nozzle. The T3 model has two mobile nozzles of 1 and 1,25 mm. Switching between chambers is made thanks to a pneumatic valve. All models have interior LED lighting. Dimensions, appearance and finish are similar.**

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Self-adhesive protective covering.  
 Packs of 10 units.

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MICROBLASTING MODULE (Pressure reducer and pedal)

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It is supplied with 1,25 mm nozzle (0,75 mm or 1 mm optional). The machine also has a pressure reducer, operation pedal and pressure gauge, so it is precise and safe to use. The support is made in stainless steel sheet.

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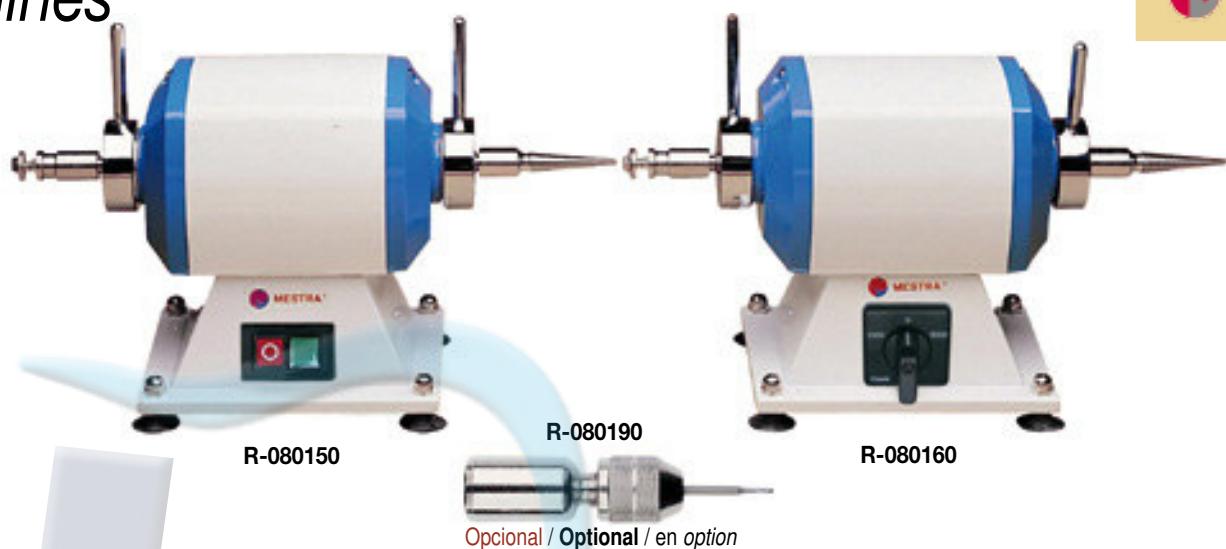
The MESTRA polishing lathe has a modern and elegant design. It is made according to the most modern concepts ensuring high quality and safety. The motor housing is made of extruded aluminium tube (special section). Outside diameter is 125 mm. Conic ends allow perfect sight of working area. Large base resting on 4 suction pads for greater stability.

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## PORTA FRESAS DRILL RACK PORTE FRAISES

	(L) 70 x Ø 18 mm		0,083 kg
--	------------------	--	----------

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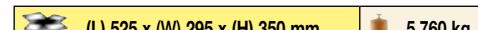
Splash protection guard provided with light and adjustable visor.

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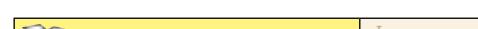
Totally made of rubber preventing any workpiece breakage due to collision with the guard.

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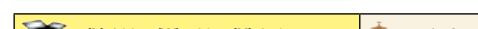
Blind. No hole for air extracting hose.

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For polishing and grinding metals, resins, etc. using dry or wet methods.

**POWER:** 500 W, 2-speed, maintenance-free motors

**QUIET:** noise level 48 dB, vibration-free. **LEAK-TIGHT:** sealed shafts to prevent water and pumice from getting in.

**SAFE:** Safety switch to prevent accidental start-up after power failures, thus removing any risk of injury.

**ERGONOMIC:** Built to allow free arm movement and comfortable work in standing or sitting position. Lever type extractors for simpler brush-holder removal. Robust, heavy construction (12 kg.) and wide base on non-slip rubber feet for maximum stability and steadiness.

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Frame made entirely of stainless steel, a non corrosive, easy-to-clean material.

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The MESTRA polishing box is a compact, functional machine made entirely in stainless steel.

It has two rubber screens in the polishing area to collect heavy particles. It is equipped with lighting LED and suction.

It meets the most stringent regulations regarding safety and environment:

- General protection by means of fuses.
- Safeguard against untimely start-ups in the absence of voltage by means of safety switch with manual reset.
- Silent self-contained suction unit (67 dB) with collection of particles by filter.
- Lighting by means of two antifatigue lamps.

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**Polycarbonate guard to prevent any splashes getting out of the polisher. 85° swivel.**

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**The waste bag is easy to remove, and features three different layers: retainer, filter n° 1 and filter n° 2.**

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**The box has 2 switches, plus a safety one. Switch nº 1 runs the motor at 1500 or 3000 rpm. Nº 2 works the light and suction device. The safety switch stops the motor so that it will not start up unexpectedly after a power cut. This button must be reset manually.**

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**Asynchronous, large flow-rate 500 m<sup>3</sup>/h, 260 W suction motor.**

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**Removable, washable guards made of flexible synthetic rubber to cushion workpieces against breakage if dropped.**

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**The suction system uses vacuum pressure and takes waste directly to the waste bag.**

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**Powerful, two-speed 500 W motor designed especially for the box. Fully sealed and with insulated shaft to prevent any water or pumice from entering, the motor also incorporates lever type extractors to facilitate brush entry.**

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**Lighting made by energy saving LED lamps with a colour temperature of 5600-6000 °K, similar to sunlight.**

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Similar characteristics to those of the Polisher Box with Aspiration R-080500 but without aspiration motor nor filtering system, so it is necessary to connect it to an external equipment of centralized aspiration. This machine is supplied with two inlets for tubes of Ø 40 mm. at the rear.

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This box with exhauster for polishing gives excellent value for money, and provides an economical way to adapt laboratory polishers to safety standards. It features an exhauster and filter unit, lighting system, polycarbonate protective screens and a Schuko type power connection with a manual safety reset to prevent unexpected start-ups (EC standards). The box body is high-strength polypropylene, combined with a steel chassis to form a highly robust unit. This also eliminates the corrosion problems to which machines of this type are prone. The body can also be used with all models of polisher, though it is intended principally for use with MESTRA™ models. Its ergonomic design makes polishing and changing the filter simple and comfortable. The machine is supplied with two rubber screens. As an option, it can be supplied with R-080150, R-080160 or R-080162 polishers screwed to its base.

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**There is an electric point in the front part of the box to connect it to the polisher. With this simple manoeuvre the machine is ready to be used.**

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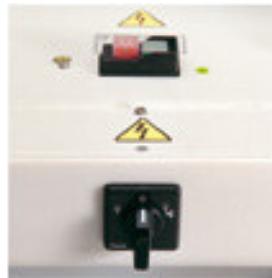
**Working area is illuminated with long lasting fluorescent tubes.**

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**It is easy to select the working mode of the box by means of an easy access switch: illumination + aspiration or just illumination. The box also includes a security switch with manual switch-on after disconnection. This makes possible to adapt old polisher models to safety rules.**

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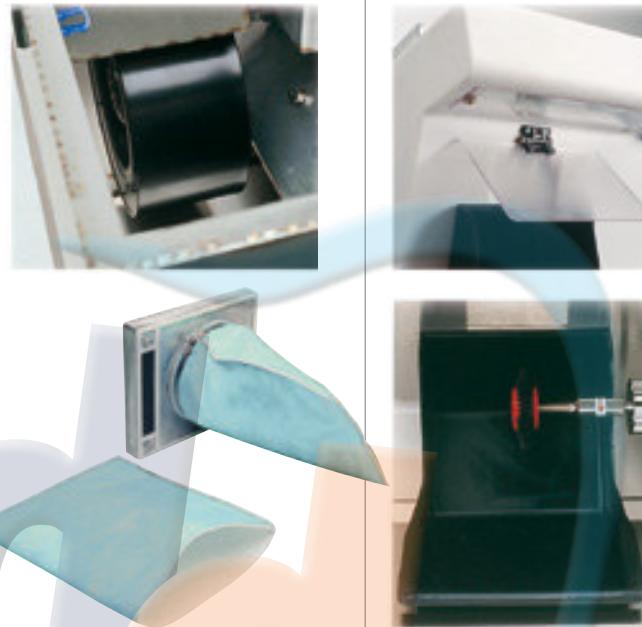
**Suction,filtering and particle collection system is the same as the one of the box R-080500.**

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**As safety element in the polishing area the machine has visors made in polycarbonate and synthetic rubber guards, easy to take out and to clean.**

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**It has similar characteristics to those of the Box with Aspiration for Polisher (R-080502) but without aspiration motor nor filtering system, so it is necessary to connect it to an outer centralized aspiration system. The machine is supplied with two inlets for Ø 40 mm tube on the back side.**

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Ideal machine for polishing in dental environments. Thanks to its small size it can be placed anywhere, taking up almost no space. Its surprising spin torque makes it an efficient tool. Speed can be adjusted between 1 000 and 7 000 rpm, so there is no risk of burning materials because excessive speed. Its wide base and the suction pads give the machine a perfect stability. It has a safety switch to prevent unexpected start-ups.

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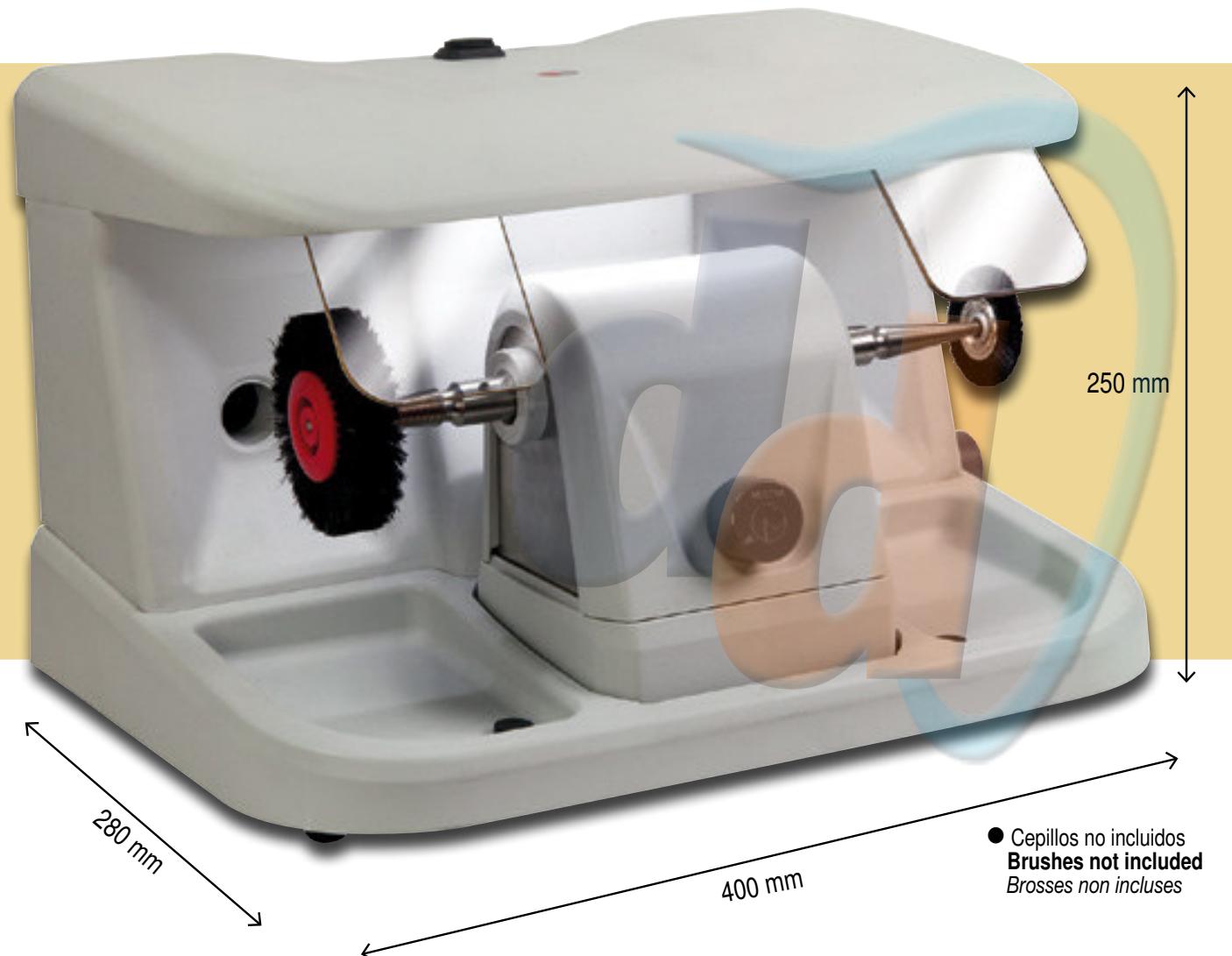
With this accessory, the Gorbea polishing box becomes a complete professional polishing box: LED lighting, suction circuit attachable to an external aspiration system (not included), trays for pumice, protection screens, electric protection against unexpected start-ups, etc. Its reduced size allows to place it anywhere. The set includes the polisher and the box.

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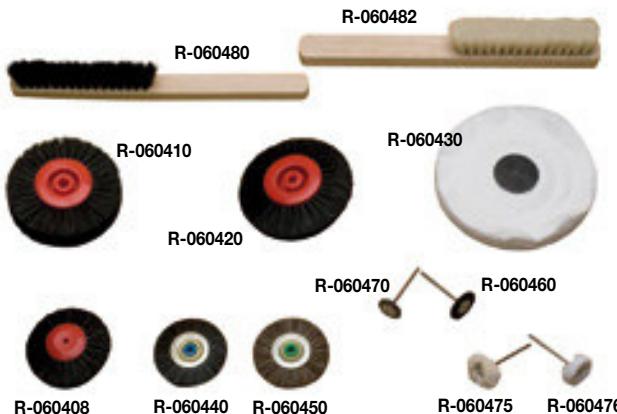
ø 75/80 brush with 4 straight rows (for going over & fine adjustment)

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ø 75/80 brush with 4 convergent rows (for going over & fine adjustment)

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R - 060408 (SET 18 uni.)

Cepillo 3 hilera convergentes ø 55 mm (para repasar y afinar)

ø 55 brush with 3 convergent rows (for going over and fine adjustment)

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ø 48 brush with plastic bushing (for going over occlusal surfaces, spaces between teeth, etc.)

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ø 48 brush with plastic bushing (for polishing resin dentures before buffing)

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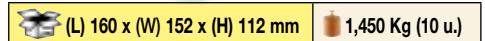
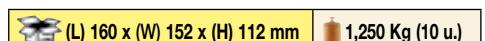
POLISHING PASTE WHITE - FINISHING METALS AND SOME PLASTICS - 115 g

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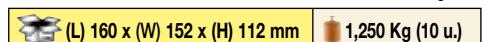
POLISHING PASTE BLUE - SUPER FINISHING METALS AND SOME PLASTICS - 115 g

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25 mm diameter and 5 mm thick, in soft or hard texture.

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And in different measures according to table below.

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The "Calypso" model is a last generation Ultrasonic Cleaner that includes the latest innovations: Sandwich type transducers, with 100W real power transmitted to the tank; working frequency of 40 KHz., representing an optimum compromise between sonority and cleaning power; electronic time control run by microprocessor, with digital display and warning acoustic signals; automatic heating system of cleaning liquid; high density polyurethane body, chemical agent resistant material; tank of 4 litre capacity and waste pipe; etc. All these characteristics are complemented by an attractive line of careful design.

**The machine is supplied with the following accessories:**

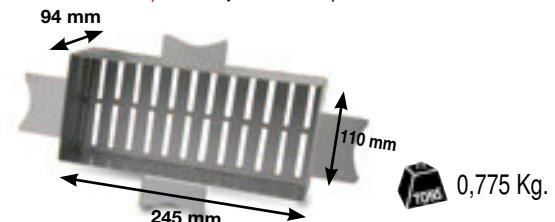
- A stainless steel basket for equipment cleaning
- That eliminates noise while working and avoids dirt coming into the cleaner
- A glass for cleaning of little instruments and its support
- MESTRA Bur-cleaner tweezers R-070520

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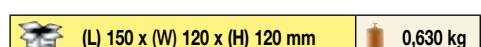
This attractively designed immersion wax heater is an essential unit for any dental laboratory. It has electronic control via microprocessor to ensure that the set temperature entered by the user is precisely maintained. The working temperature range is from 50° C to 99° C and the temperature inside the tank can be monitored in real time via an electronic display. The set temperature can be introduced by touching a switch. On start-up the control programmes a ramp up to 80° C, and holds that temperature for 10 minutes to ensure that the wax is completely melted to an even consistency. After this initial stage it goes to the set temperature, which is held until a new setting is entered by the user. Its size and sitting area are optimised to prevent overturning. The

front of the tank is smooth so that wax spills during use can be cleaned up easily.

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The MESTRA wax heater is designed to provide practical, efficient service in the laboratory. No superfluous decorative elements are provided on its front. This makes it more functional, as drops of wax inevitably fall on this area when the spatula is being inserted and removed, which would damage the controls. The MESTRA wax heater is set for a temperature of around 85 °C, which melts wax in 8 to 10 minutes.

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Very simple and easy to use. Just enter an operating temperature, the electronic control system keeps the programmed temperature to a surprising degree of accuracy and shows it on an illuminated screen. Its strong plastic frame and membrane keypad stop wax from filtering inside the appliance. The tank is divided into four independent compartments so that as many different types of wax can be heated. The teflon coating of the tank makes cleaning operations easier.

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Similar to R-080072, but unlike that has a single container.

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This machine is extremely useful when working with applications that need a water bath that keeps the water temperature precisely (polymerisation of little elements, wax sheets for biting registers, etc.). It has a 1 litre stainless steel tank and it is easy to clean. The body is made in high density Polyurethane, resistant to chemical agents. Bath temperature control is made by an electronic thermostat easy to use and precise when keeping the programmed temperature. Temperature range is from 30 °C to 75 °C. A powerful heater allows reaching the set temperature in a few minutes.

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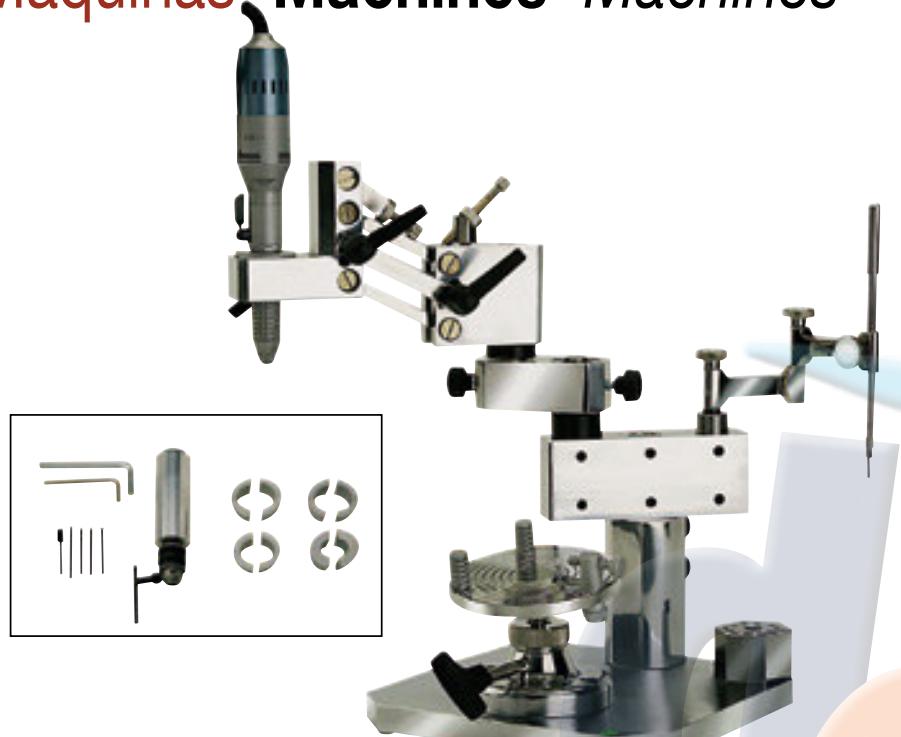
Similar to the R-080078 Water Bath, but in this model the controls allow to regulate a wider range of temperatures and the tank is teflon coated. Thanks to these two modifications, you can keep a considerable amount of melted wax in the tank, ready to be used in different lab techniques.

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Parallelometer. Wax knife

Analyser rod. Rods with 0,25, 0,5 and 0,75 ridges

Leads, Allen wrench, 5 bushings for fitting different hand tools

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Pièce à main non comprise

R - 080410

## CARACTERISTICAS / CHARACTERISTICS

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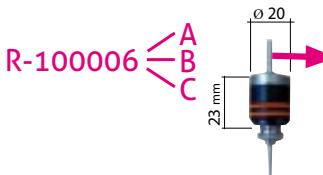
Ø 3, 2,35 or 2 mm fixing shaft (diameter to be chosen when ordering). Ideal to be fitted in a parallelometer, micro-drilling support or similar device for modelling wax pieces. It is factory supplied with a complete tip set and an electronic control box. The control box allows an accurate adjustment of the working temperature of the tip by showing it on the digital display.

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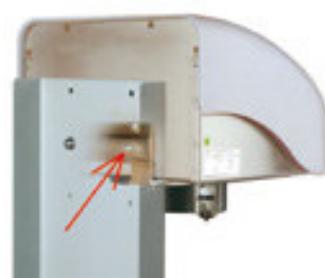
**It is equipped with a powerful and solid step-down motor with nylon gears (less noise and friction), and a two bodies high vacuum pump. The Koala mixer makes mixing of investment, plaster, and other materials possible. Thanks to its electronic control it is possible to easily introduce the operating time and also to choose cycles of only vacuum, or mixing + vacuum. With the programme switch cycles up to 3 steps can be defined. These are very useful when working with certain techniques. This machine is supplied with all necessary anchorages in order to fit it to the wall and with two bowls of 250 cc and 550 cc. There are also other accessories available like a fastening arm for bowls for mixing without vacuum technique and a base that changes it into a table model.**

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Another recommended accessory for those who use non-vacuum mixing techniques is the fastening arm (Ref. 080522). The arm system keeps the bowl joined to the mixing body when working in non-vacuum cycles. It is possible to select height between two different bowl sizes (250 cc./ 550 cc.) by just turning it.

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Koala Vacuum Mixer has a high performance pneumatic cartridge type filter that protects mechanical elements against dust and dirt.

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Replacement or cleaning of filter is easy. Access is immediate and the machine does not have to be dismantled. No tool is needed.

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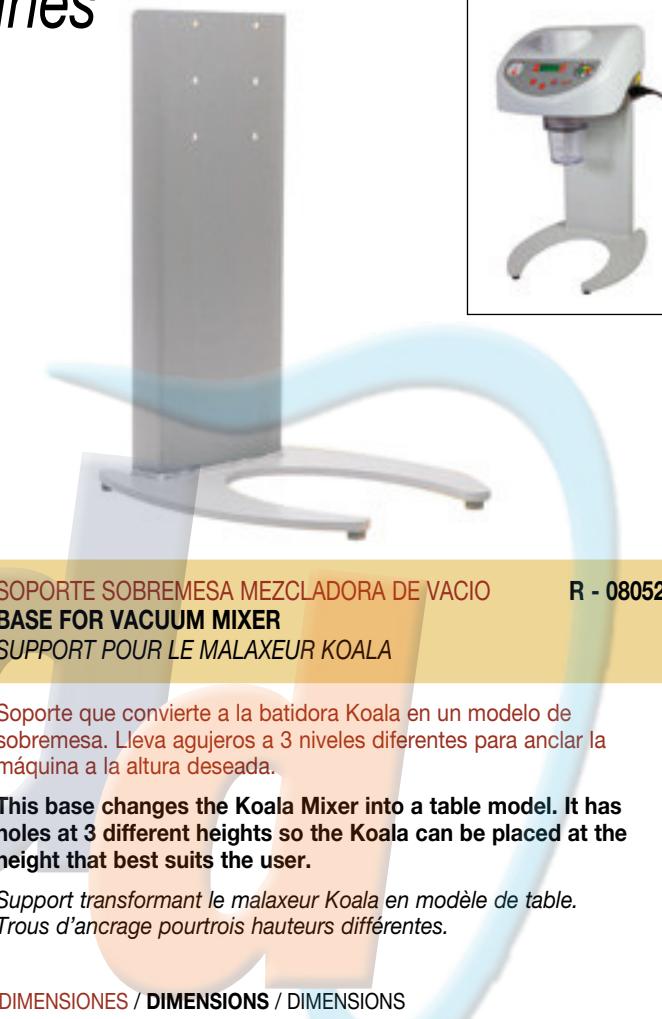
In case the Vacuum Mixer stops due to electrical supply breaking, the bowl can be released from the mixing body (emergency manoeuvre), pressing the unloading valve on the filter casing.

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This base changes the Koala Mixer into a table model. It has holes at 3 different heights so the Koala can be placed at the height that best suits the user.

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Koala Vacuum Mixer has one of the largest range of bowls available in the market, with capacities of 250 cc.; 550 cc.; 750 cc.; 1.000 c.c.; 1.250 c.c.; and 230 cc. (specially designed for alginate mixing). All of them are made in resistant clear polycarbonate. R-080529 Stainless steel.

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Economical and useful vacuum mixer. Just place the bowl and press the operation key and the mixer will start up. In a few seconds the mixture will be ready. As there are three versions available, you will find the model that best suits your needs. Despite the differences in their control system, all the versions share the same engine-pump-chassis structure –of well known solidity and reliability.

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**The IRIS family consists of three models: the IRIS II, a simple operation mixer; the IRIS II Evolution, a high performance mixer; and the IRIS II Advanced, a sophisticated mixer which can program and perform multiple functions. All the models have an attractive, practical and ergonomic design.**

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**The bowl is comfortably and smoothly coupled to the machine. To release the bowl, just press an easily accessible button.**

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**The main feature of the Iris II mixer is its simplicity. Just place the bowl, and select the mixing time. The machine will perform the cycle immediately. The Evolution model has electronic controls which allows adjusting the operation time and the rotation speed between 100 and 400 rpm. The Advanced model, apart from these functions, allows programming pre-emptying and pre-mixing times, turbo function, post-emptying time, alarms, rotation reversing... With these features the mixing can be optimized using the wide range of materials in the market: coatings, type II, III, IV plasters.**

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**The electronic controls in the Evolution and Advanced models allow the user to design, store and use his own programs. Besides, in the Advanced model the electronic controls can program different alarms and many other functions. Comfort and performance need not be at odds.**

**Both models are factory supplied with all the necessary elements to fix them to the wall. A stand is also available. The Iris 2 mixers are factory supplied with a 550 cc bowl. However, there is a wide range of bowls available to adapt the mixers to all sort of needs.**

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Attractive and reliable gel mixer featuring state-of-the-art technology: tank heating by lateral and bottom resistors, which prevents gel from burning because of too high temperatures; belt driven system which allows an extremely smooth and silent operation; oxide free casing, made of stainless steel and polyurethane; high capacity (7 litre tank); state of the art electronic controls allowing multiple adjustments... A complete machine designed to work without problems for many years.

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## The Cooling

If we want to obtain ready-to-pour gel from solid gel in the shortest possible time, it is easy to understand that a quick cooling is as important as a quick heating. The engineer team in **MESTRA** has carried out a smart design work: the power of the ventilation system has been doubled compared to the GEL-BAT 6 model, and the design of the hot air extracting system allows to obtain gel in an ideal pouring state in a record time. In fact, for a given amount of gel, the cycle time has been reduced almost a 35% compared to the time in the GEL-BAT 6 model.

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Easily operated tap. Made of non-stick thermo-plastics

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## The cold start

The “motor/transmission/blades” cinematic string of the ANETO gel mixer has been sized bearing in mind this operation: The controls protect the motor, and set the exact parameters (intensity, speed and torque) during the start and prevent sudden movements that might cause damages or accidents. In addition, elastic relief-elements have been placed between the different pieces. Such a prevention guarantees a long working life of the mechanical elements of the machine.

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The sealing lid triggers a safety sensor that prevents the operation of the engine and the resistor while the lid is open.

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## The controls

The sophisticated features described above are completely useless if they are not strictly controlled. Hence the decisive role of the controls in the good operation of the machine. The motion of the engine is completely monitored: the starts, stops, stresses and overloads are checked every few thousandths of second, and the resulting controlled motion never forces the transmission system. And this turns into a smooth operation and a long and excellent working life of the machine. The temperature is also thoroughly controlled to co-ordinate the heating of the two resistors as efficiently as possible: the heating takes place at the highest possible speed for the gel not to get burnt, and the cooling takes place very quickly to finish the cycle in the shortest possible time. After all, the secret to achieve an impeccable operation of the machine lies in wisely combining the mixing and the heating. Needless to say, apart from monitoring these two important parameters, the controls of the ANETO gel mixer perform many other important functions: programming of gradients and drops in the cycle, programming the waiting times, synoptic control of the cycle; programme storage; alarms, etc. All these tools are easy to configure, and help adapt the machine to the user's likes and needs.

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MESTRA gelatine mixer, made of stainless steel. The control microprocessor provides the user with a wide range of options, such as that of preprogramming gelatine temperature for a given time. Example if it is 7:00 pm, and we want the gelatine to be ready for pouring at 8:00 am, all we have to do is program "8 am" on the control. Also we can change from pouring temperature C-1 (75°C - 95°C) to maintenance temperature C-2 (35°C - 65°C), or vice versa, simply by pressing the button. Because its torque is high, the geared motor can be easily started even if the tank is full (the gelatine cut into pieces).

It has all safety features required by the CE mark, including:

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**Easy-to-use tap made of non-stick thermoplastic.**

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**Its high-torque gear-motor enables the unit to start up with solidified gel.  
A heat relay safety system protects the motor against overheating.**

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**Vane inclination and distribution designed for even gel mixing.**

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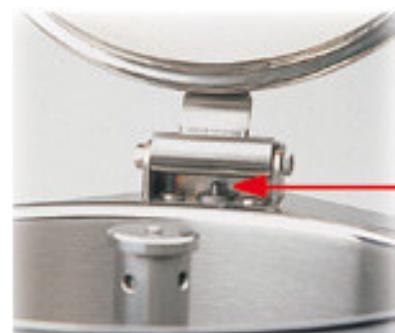
**A safety switch prevents the motor and heating element from working while the lid is open.**

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**Forced-draught tank cooling on reaching setting C1.**

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**Extremely high performance pre-heating furnace family, composed of three models with different muffle capacity (2,5, 5 and 10 litres). Made of stainless steel, the most convenient material for the extraordinary thermic requirements this kind of devices has to bear. The high quality of the muffle ensures an exceptionally long working life, even when continuously working at temperatures near 1100 °C. When the thermocouple fails, the electronic control will announce it. This element is very economical and easy to replace by the user himself. On the other hand, the electronic control allows you to programme cycles composed of multiple parameters; you can define ramps and set point temperatures, times, heating in degrees per minute, °C. The memory can store up to twenty different programmes. The furnace is factory supplied with a ceramic tray.**

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HP family muffle chassis is entirely made of stainless steel, heat degradation resistant material and easy to clean.

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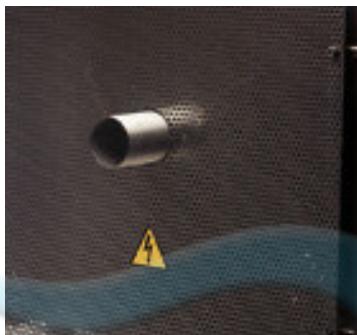
**The muffle is made in ceramic material with heaters wrapped along the upper, lower and on both sides. This way, there is a uniform interior distribution of temperatures. The exterior sides of the muffle are thermally insulated by zirconium silicate.**

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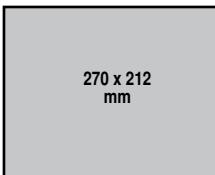
Gas flow inside the muffle has been carefully studied. There is an air inlet at the door and a ventilation pipe at the back, both with calibrated sections. This way air inside the muffle is renovated without reducing thermal efficiency of the oven nor producing temperature differences.

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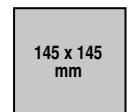
A practical socket allows the synchronised operation of the oven together with an extraction unit. Automatic disconnection can also be programmed between 75° C and 850° C. When temperature is over 850° C, the control automatically disconnects the power supply.

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**Magnetic closing system is easy and simple. A safety sensor detects door opening, and disconnects heaters while it is opened.**

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**concept** HP family muffle chassis is entirely made of stainless steel, heat degradation resitant material and easy to clean.

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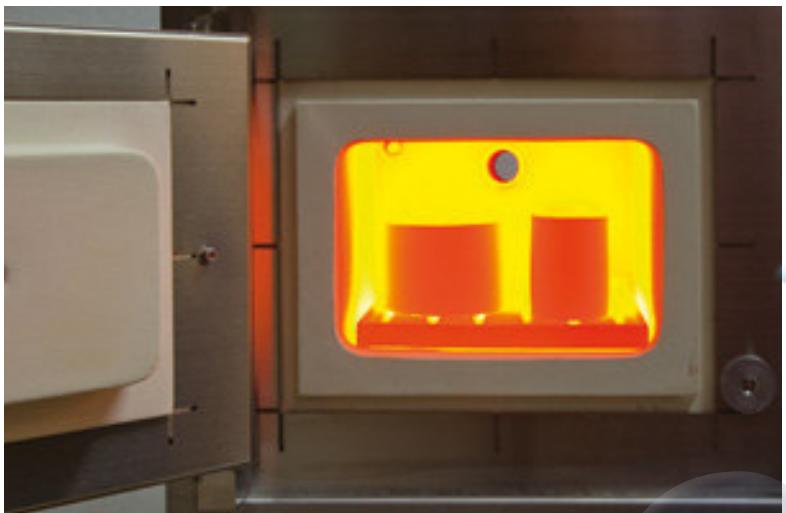
The "Concept" controls family has been developed as an economical alternative to the MESTRA's standard electronic controls. The only difference between a "Concept" product and a standard one lies in the controls; all the other components and functions are identical. The performance and features of the "Concept" controls meet perfectly the needs and expectations of most of the users, but at a very attractive price. For this reason, it is recommended for those users who do not need very high levels of performance. Apart from the price, its main advantages are its ease of operation and programming. The "Concept" line is available for all the MESTRA furnace range.

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**As a core element of the furnace, the muffle has received special attention from our design team: the electric resistor covers the four sides of the inner chamber, which achieves a more uniform heating. Besides, it protects the chamber from occasional friction or scratches, and gives it an exceptional thermic insulation. If we take into account that its inner size and shape have been optimized to take the maximum number of cylinders, we will understand the high amount of energy saved.**

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## ERGONOMICS

Twin display screen: one for temperature, the other for time. Graphic showing temperature increase and holding, with indication of what process is ongoing.

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It is perfect for those works that do not require a classical programming cycle of ramps and plateaus. Just select the set point temperature, and the furnace will start heating directly until this temperature is reached. Then, this temperature will be kept for a time programmed by the user. Its digital controls ensure an amazing temperature accuracy and prevent the effects of thermal inertia. Besides, it features a connection for an external catalytic converter, and this element will operate from the beginning of the cycle up to the stopping temperature programmed by the user. The size, operation and other features of this furnace are similar to those of the HP-50 model. The only different feature is its controls. Tank capacity: 4,8 litres (4 to 6 casting rings Ø 80).

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Installing this device, you succeed in getting rid of furnace gases generated in the muffle in a continuous and smooth way. It is completely made in stainless steel and its installation is easy. If it is connected to the MESTRA HP 25, 50, 100, Furnace socket, the extractor runs with programmed disconnection between 75 °C and 850 °C.

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Simple device attached to the flue outlet of any preheat oven to remove gases and combustion residue that are generated in the oven as a result of the combustion of waxes and other materials. A forced ventilation system eliminates the gases produced inside the oven and takes them to the extractor hood located on the exterior. As they pass through the apparatus, the gases and combustion residues are reheated and catalysed, being reduced to CO<sub>2</sub> and H<sub>2</sub>O. The use of this device eliminates a good part of the bad odours and harmful products (when inhaled) that are generated during preheating processes. The catalyst is specially designed to work with the ovens of the HP family by MESTRA, but it can also be coupled to any other preheat oven with an outlet pipe of less than 38 mm Ø exterior.

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**Easy to handle and high performance furnace:** it can maintain a steady temperature of 1600 °C and reach peaks of 1650 °C. The electronic controls allow programming a heating cycle with multiple rises and plateaus. It is surprisingly accurate when rising and maintaining the temperatures. Due to its reduced size (similar to that of a pre-heating furnace) it can be placed anywhere in the lab. However, the chamber is big enough (100 x 100 x 100 mm) to contain the production of a whole day in a medium/big lab. Its low power consumption makes it an economical and environmentally-friendly furnace. It is an ideal element for those labs looking for a highly reliable equipment at a suitable price; even for those labs that already have one, but look for an economical option in case of failure in the main equipment.

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Fully automatic, compatible with the different types of porcelain in the market. It includes a high-vacuum pump. It features a wide display with complete information about the process given by the programme. It can store multiple programmes and modify parameters such as temperature and drying time, heating speed, maximum temperature, maintenance and cooling time, vacuum times and levels, etc. All these parameters allow defining the ideal cycle for each material and obtaining a perfect result. The temperature can be calibrated in order to improve the accuracy. The furnace has different alarms to warn about any anomaly (breakage of thermocouple or resistor, failure in the vacuum-system, etc.)

The furnace is factory supplied with the following accessories:

- 1 vacuum pump
- 1 heating platform
- 1 calibration kit
- 1 tray

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**Save wax!**

Wax is recovered in the lower tray, and it can be reused.

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Practical device designed to remove the wax from the casting rings before inserting them into the pre-heating furnace. This simple and quick operation prevents the fumes and bad odours produced by the wax when burning inside the furnace. Its operation is simple: just fill the tank with water to the tray and place the casting rings inside. A powerful resistor will heat the water up in a few minutes until the wax is melted by the action of the steam. By means of a thermostat the working temperature can be regulated at will. Besides, the machine can be used for other operations that require a tank full of warm water at a given temperature, for instance thermal-polymerization of small elements. The machine is made completely of stainless steel, material which ensures a quick cleaning and a superb resistance to corrosion. A basket for the casting rings and a stainless steel lid are included.

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Plaster and water dispenser. Just press a button, and you will obtain a dose ready to be mixed. Besides its speed and precision, another advantage of this product is the saving of materials: when you dispense by hand, the trend is to increase the dose. However, with the Boomerang dispenser you will always obtain the exact dose, and the subsequent saving. According to our experience and estimates, the saving would be between 25 and 40%, quite important amount that will enable you to pay off the machine in a very short time. There are two versions available: the "evolution" version, which works without trouble with Type II and Type III powder, and the "precision" version, featuring a scale and recommended for working with Type IV powder. Both models have electronic controls that allow to alternate different types of powder by just replacing the tank, to programme cycles, to work with fixed proportions, and so on. They are factory supplied with a rubber bowl to mix the plaster and a plastic spatula. Model R-080592 also includes an extra plaster tank (to alternate different types of powder) and a bracket to fix it to the wall.

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Easily removable powder tanks: it simplifies powder-filling and replacing plaster cartridges of different types. With the Boomerang dispenser a few seconds will be enough to replace the tank, and, therefore, the product to be dispensed.

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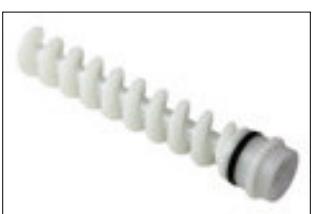
Smartank dispensing system: the built-in dispensing mechanism is integrated in the tank, so that in case of obstruction or wear, it can be repaired in a few minutes by the user himself. The mechanism consists of a spindle, an impeller and an actuator, all of them easily accessible.

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**Accuracy:** Accuracy is one of the most surprising features of the Boomerang dispenser. Deviations are small and fall easily within the tolerance range of dental plasters.

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**Intuitive multifunction button:** a keystroke is enough for the dose shown on the display to be dispensed. If the button is pressed repeatedly, the dose shown on the display will be multiplied by the number of keystrokes. If the button is kept pressed, the machine will dispense powder continuously as long as the button is pressed. Next the machine will dispense the proportional amount of water.

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**Style:** The Boomerang dispenser has an attractive design, combining practicality and ergonomics. As a rule in all MESTRA products, toughness and reliability are guaranteed. And perhaps the most astonishing feature of the Boomerang dispenser is its price, much lower than it could be expected in a machine of this quality level and features. This model is factory supplied with two powder tanks and a bracket to fix it to the wall.

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**Style:** The Boomerang dispenser has an attractive design, combining practicality and ergonomics. As a rule in all MESTRA products, toughness and reliability are guaranteed. And perhaps the most astonishing feature of the Boomerang dispenser is its price, much lower than it could be expected in a machine of this quality level and features. We encourage you to contact your MESTRA dealer. You will be pleasantly surprised.

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Carefully designed to facilitate the everyday work of the dental technician: thanks to its compact size, it can be placed everywhere, the filters are easily replaced without squatting, the motor can be replaced **in less than 1 minute\***, optimum noise isolation, it can accurately detect up to three micromotors, wheels to move it easily, etc.

Electronic control with protection system for both overloads and aspiration inlet obstructions. It also has a visual indicator of full filter level. The Aspiration Unit allows working in "synchronized start-up" mode with up to three different machines (sandblasters, micromotors,...).

The detection operation and the adjustment of the micromotors is carried out by the electronic controls, which allows adapting the inlet levels to any micromotor, even to the brush-less type micromotors. All these features together with its excellent soundproofness, attractive line, and of course being robust and reliable as all MESTRA products, makes Eolo Plus an interesting purchase option.

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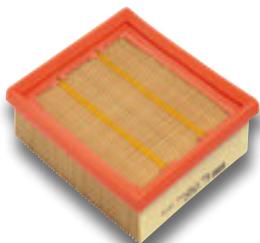
**The upper lid of the Eolo Plus aspiration unit can be easily removed, giving full access to the filter, allowing to replace it without uncomfortable positions. The hook-type lock at the rear frees the lid and the filter.**

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**The Eolo Plus aspiration unit also features a second filtering stage consisting of a high performance**

**filter. The combination of the two filtering stages allows the aspiration of all types of materials, included abrasives such as aluminium or silicon oxides.**

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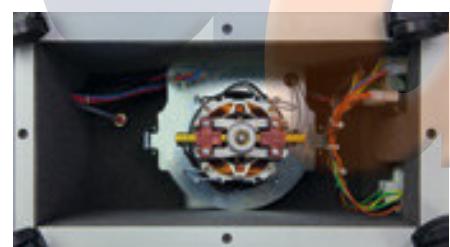
**A set of practical wheels allows to move the unit around in order to facilitate operations such as filter replacement, to share it with different work places, etc.**

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**The aspiration motor has a power output of 900 W. It is placed in the lower part of the chassis, completely encapsulated in phono-absorbent material to reduce the noise level.**

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**At the rear of the Eolo Plus aspiration unit there are 3 power outlets to connect different devices, fitted with a remountable thermal cover.**

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**The controls panel of the Eolo Plus aspiration unit is at the front of the machine, allowing easy access to the controls. It is a synoptic type panel and, therefore, the combination of lights,**

**diagrams and buttons allows an intuitive operation of the machine.**

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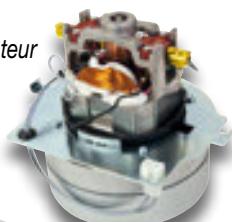
**The first filtering stage consists of a high capacity bag filter.**

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**It is a simple device to be connected to the inlet pipe of a conventional Aspiration Unit for pre-filtering dust particles. The air circulates along a circuit that makes the dust particles fall into an inner tank. More than 99% of the aspirated dust and dirt is removed, reducing the risk of obstruction of the aspiration pipes and making the working life of the aspiration filters considerably longer. The dust tank is easily accessible and, therefore, cleaning the device is a comfortable and simple task. The device features anchoring elements especially adapted to MESTRA's EOLO-PLUS aspiration, with which it makes up a high performance compact unit. CICLÓN is built in high resistance polypropylene.**

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The installation of the unit is very simple. It is simply fixed to the wall or a holder with three bolts. The intake pipe can be positioned to either side, left or right. The unit is factory supplied with all the accessories required for its complete installation: connecting hoses, intake adaptors, holding plate, fixing bolts.

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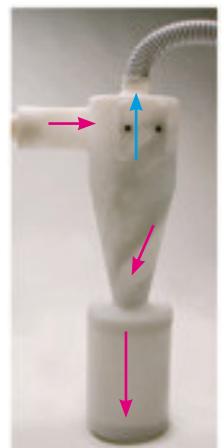
The "Ciclón" Dust Decanter features a high capacity tank (3.5 L). Emptying and cleaning the unit only takes a few seconds, due to its excellent accessibility and simple threaded lid.

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The operating principle of the "Ciclón" Dust Decanter is a very ingenious one: the aspirated airflow circulates within the machine's body along a helicoid. The own weight of the dust prevents it from moving up toward the outlet pipe and makes it fall to the tank.

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The "Ciclón" Dust Decanter works perfectly together with different machines and in different situations, from a simple workbench to a sandblaster. Its small size and compact design allow it.

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**It is cheap, easy to use and has an attractive design. It has a powerful aspiration motor with automatic connection when the sheet holder goes down. The carbon heater allows a progressive and uniform heating of the material that is going to be formed. It can be used with square sheets (standard 127 mm side). The machine is supplied with a plane model holder and with a basket that can be filled with grit (a can included with the machine) so forming depth is reduced.**

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**Standard square thermoforming sheets of 127x 127 mm. Made of high quality plastic with FDA approval and CE marking certifications. There is no soot nor fumes during the heating. As they do not contain recycled materials, they do not tear while subjected to vacuum. They are available in a wide range of thickness and types.**

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**Nobody questions the need and importance of steam jet cleaning in the Dental Lab. Nobody questions its consequences either: dirt in sinks, air moisture, condensation in walls and floors...**  
**And all this leaving aside the high price these kind of devices used to have up to now. In **MESTRA<sup>®</sup>** we have worked hard to make these troubles simply disappear by using the Altamira Steam Cleaning Cabin, the indispensable complement for any steam cleaning device.**

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**It is manufactured on a strong polypropylene framework, and has a wide window to access the chamber. All the walls are smooth to make cleaning easier. The device features a deflecting frame carefully studied for the steam to remain inside the chamber, with no leaks.**

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**Lighting is another of the most carefully studied aspects of the Altamira Steam Cleaning Cabin: two powerful low-energy LED bulbs give a perfect lighting to the pieces steamed.**

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**The primary filter is located inside the main chamber, so it is easily accessible. No tools are required to clean or replace the filter. The second filter is located at the rear of the device, and it requires even less maintenance than the former**

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**A practical sight-level glass shows when the tank is full, so you can empty it via a comfortable hose located at the rear of the device.**

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**The heart of the Steam Cleaning Cabin is a powerful high-flow fan, which easily aspirates the steam accumulated inside the main chamber. It is powerful and silent.**

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**The Altamira Steam Cleaning Cabin is a new approach to steam cleaning in the Dental Lab, attractive and at a low price. Of course, with the usual quality level and after-sales service of all the **MESTRA<sup>®</sup>** products.**

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# Máquinas Machines Machines

**BOX APPLICACION SPRAY (para aspiración externa)** R-080602

**BOX FOR SPRAY APPLICATION (for external aspiration)**

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The Altamira Box is specially adapted to apply the anti-reflection spray, before scanning models. The unit requires an external suction system (not included) that aspirates a powerful air flow out of the chamber, so preventing the spray dirtying or polluting the air in the working area. Besides, it is very useful to clean the metal dust remaining in the discs of the milling machine.

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**Reliable, safe and resistant. Tank capacity of 3 litres and working pressure of 3 bar. The tank is completely made from high-quality stainless steel.**

The powerful heater means that working pressure can be reached in just a few minutes. The level of water in the tank can always be checked through a peephole.

To avoid calcification in the boiler and extend the working life of the appliance, the machine must be filled with decalcified water, or preferably distilled water.

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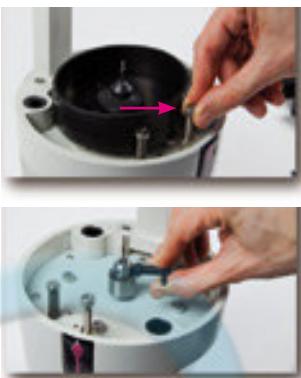
**Easy to handle the machine allows to drill precision holes in the lower part of the model. Changing and adjusting the bit is an easy and intuitive operation. The machine works silently and without vibration. The laser in the upper part of the machine allows to locate exactly the place to drill. Its size, simplicity and ergonomics make it the ideal device for the technique of individualizing pins. The machine is factory supplied with 2 burs: 1.85 mm and 1.95 mm.**

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**Details are very important when designing a good pin drill: the drill-anchoring system has to be easy to loosen. In the case of the **Atlas** pin drill, a few seconds are enough to replace the drill or adjust its height. The table can easily regulate the depth of the drill: you just have to turn a screw to adjust the height at will.**

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**In the **Atlas** model the table is easily dismounted without any special tool, and it gives access to an inner container to collect the dirt. Once removed, you can clean the container with water and soap before placing it again.**

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**Another significant feature is the smooth operation of the drill. Unlike in some other machines, in the **Atlas** pin drill the table is guided by axial bearings and return springs. The result is astonishing: the touch is firm and accurate, with the exact amount of resistance and elasticity, and without the slightest play.**

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**Ideal tool for working with the individualization technique. Its 0.3 mm thick diamond disc allows thoroughly cutting a plaster model and separating it in individual pieces. Thanks to its outstanding accuracy, there will be no fitting problems at all during the operations after the individualization. In this machine special attention has been paid to safety: the disc has a cover that keeps it always careened, the model holder has a powerful grip to the table, the motor has a double main switch which requires using both hands to start up the unit. All these measures make almost impossible an accident, if the machine is operated according to the safety measures. Moreover, its size is reduced and the table very comfortable to use, and it does not take up too much space, it allows all kind of turns and angles to cut the model in the ideal place. Besides, it has a pleasant look and an attractive price.**

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**This Curing Light is completely made in stainless steel. It scans ultraviolet light among 280 and 550 nanometers, covering most wavelength used in curing dental materials. It is a General Purpose Machine, essential in Dental Clinic and Laboratory. Its main features are its time electronic control with digital display, acoustic signal of end of cycle and encoder type unique control programme; optimum access to the inside chamber of the machine by a door on the top; automatic warning of lamp change (each 500 working hours); revolving support so that light is better distributed (135 mm Ø); panoramic view from porthole on the top of the machine; and security switch that switch off lights when the door is opened. Sun Beam Curing Light is supplied with a practical support for prosthesis (MESTRA R-080550-50) and a bag with fixing pins.**

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A rotating plate moves the material to be cured. The light bath reaches all corners, and distributes uniformly.

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The curing chamber door is on the top section of the unit. Full accessibility. Two-tone lamps light up the inside of the chamber.

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A safety switch turns off the lamps if the chamber door opens accidentally while the unit is operational.

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The curing process may be easily supervised through a spy "port-hole". The viewing angle covers the entire chamber.

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It is cheap but sturdy and reliable. It has an illuminated chamber with four fluorescent compact lamps (total light power 9 W x 4= 36 W) from 400 to 550 nanometers. It has a wide interior tray to place up to four models to polymerise at the same time. Operational time is selected with a timer from 0 to 15 minutes. When the cycle finishes, the machine emits a warning acoustic signal.

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**Stainless steel chassis, oxidation and chemical agents resistant material and easy to clean.**

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The MESTRA interior trimmer is carefully designed to facilitate trimming work in dental laboratories. It comes in a compact, fully stainless steel built unit which is easy to clean and totally corrosion-resistant. Its large sitting surface and non-slip rubber feet make the unit stable and prevent vibrations or unwanted movements during use. Its powerful 1/3 HP, 3000 rpm motor ensures optimum cutting power and smooth, quiet operation. The trimmer is designed to work with 6 mm shank diameter drills. The work table is perforated to drain off the dust caused during denture trimming, and has an outlet tube which can be connected to an external exhauster. At the rear there is a Schuko type power connection which is very useful when an exhauster is used. The interior trimmer has a safety switch to protect against unexpected start-ups and a drill protection screen.

It meets EC regulations, and is supplied with one 12-tooth hard-metal drill.

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**3000 rpm 1/3 hp Motor.** Its power guarantees the impossibility to brake the bur when cutting. Motor shaft protection retainers prevent dust entry.

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Bur protection prevents possible damages to people when the machine is left operating by mistake. When working, its gentle activation does not imply any obstacle.

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**Safety switch in accordance with Machine Security rules.** If there is an electricity supply cut, a manual switch on is needed.

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**There is a practical power point at the back of the machine where an external suction unit can be connected. It will get into work as we activate the interior trimmer main switch.**

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(L) 520 x (W) 290 x (H) 360 mm

12,500 kg

A pipe at the back of the working table permits the way out of generated dust. This machine is supplied with a nylon adapter for 35 mm diameter suction pipe.

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These burs are made of tungsten with grooving serration and are available in two different outlines: cylindrical and rounded conical. The rounded conical bur is supplied with three different kinds of serration: nº 5X (standard), nº 3 and nº 6 (fine). On the other hand, the cylindrical bur is supplied with serration nº 3 and nº 6. Dimensions: head diameter 12,5 mm; total length 77 mm.

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The MESTRA electrolytic polisher is designed specially for dental laboratories. Stainless-steel construction and large tray capacity. Up to three skeletons can be polished simultaneously. Equipped with a timer, it allows polishing to be programmed from 0 or 30'. It includes potentiometer, ammeter, thermal safety device, and visual function indicator. A highly efficient unit.

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Dental polyamide injecting machine of compact design. It can inject 7/8" (22,5 mm) cartridges, but there is a version that injects 1" cartridges, available upon request. The electronic controls allow operating in automatic and manual modes, and thoroughly adjust the injection temperature, so achieving perfect results. It features a pneumatic system to cool down the piston in order to shorten the cycles. The thorough adjustment between piston and casing prevents seizing up in case of breaking the cartridge.

The MESTRA polyamide injector can be used in different positions (hanging on the wall, lying standing on the table...), taking minimum space. Factory supplied with two aluminium flasks and a demolding spray.

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It has a pressure regulator and a pressure gauge.

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**Useful tool when making alginate paste in Dental Clinics. Easy operation: place powder and water in the bowl and select rotation speed button (fast/slow). While rotating, user can mix the paste with a spatula till it reaches appropriate consistency and texture. The technician controls visually the process at every stage, and this guarantees that the paste made is in optimal conditions for application. In order to make cleaning easier the bowl is detachable. It may be highlighted small dimensions, attractive line and easy cleaning-up.**

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Connect the machine to a power point of 230 V. 50/60 Hz. Press general switch at the rear.

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Place the bowl on the bowl holder. Fit bowl and bowl holder firmly and adjust them vertically.

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Turn the bowl 1/5 turn clockwise (approx 70°).

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Make sure the bowl is securely fitted to the base by pulling upwards.

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Pour alginate and water inside the bowl in the proportions indicated by the manufacturer of the alginate used.

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Press slow speed (tortoise) to pre-beat the mix with the spatula (provided with the machine).

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After a few seconds, pre-beating is finished. Then, press fast speed (hare) and mix the mixing pressing it with the spatula. The spatula should be placed almost flat against the wall of the bowl. (Mixing position- spatula).

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Once mixing finishes (after just a few seconds), gather the alginate mix around the wall of the bowl moving the tip of the spatula from the bottom to the upper edge of the bowl (Gathering Position- spatula).

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This is an economic option which can be used with the MESTRA R-060220 and R-060230 mixers. It is ideal for mixing plaster, coating, etc. Small and noiseless.  
VOLTAGE: 230 V/50-60 Hz CONSUMPTION: 0,72 Amp.  
VACUUM: 550-600 mm/hg

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Indispensable for dental clinics. Small size but practical and reliable. Low power consumption as it is equipped with a cartridge resistor of 100 W with two contact relays.

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**European Directives set “essential requirements” for the safety of persons and goods to protect users and the environment.**

**Our products are affected by four Directives:**

Directive 2014/35/UE, known as the “Low Voltage Directive”.

Directive 2006/42/CE, known as the “Machines Directive”.

Directive 2014/30/UE, known as the “Electromagnetic Compatibility Directive (CEM)”.

Directive 93/42/CE, known as the “Medical Devices Directive”.

**Our products meet the requirements of the Directives which affect them, and bear the CE mark as proof of compliance.**

**We are at the disposal of our customers to provide any information they may require on this matter.**

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